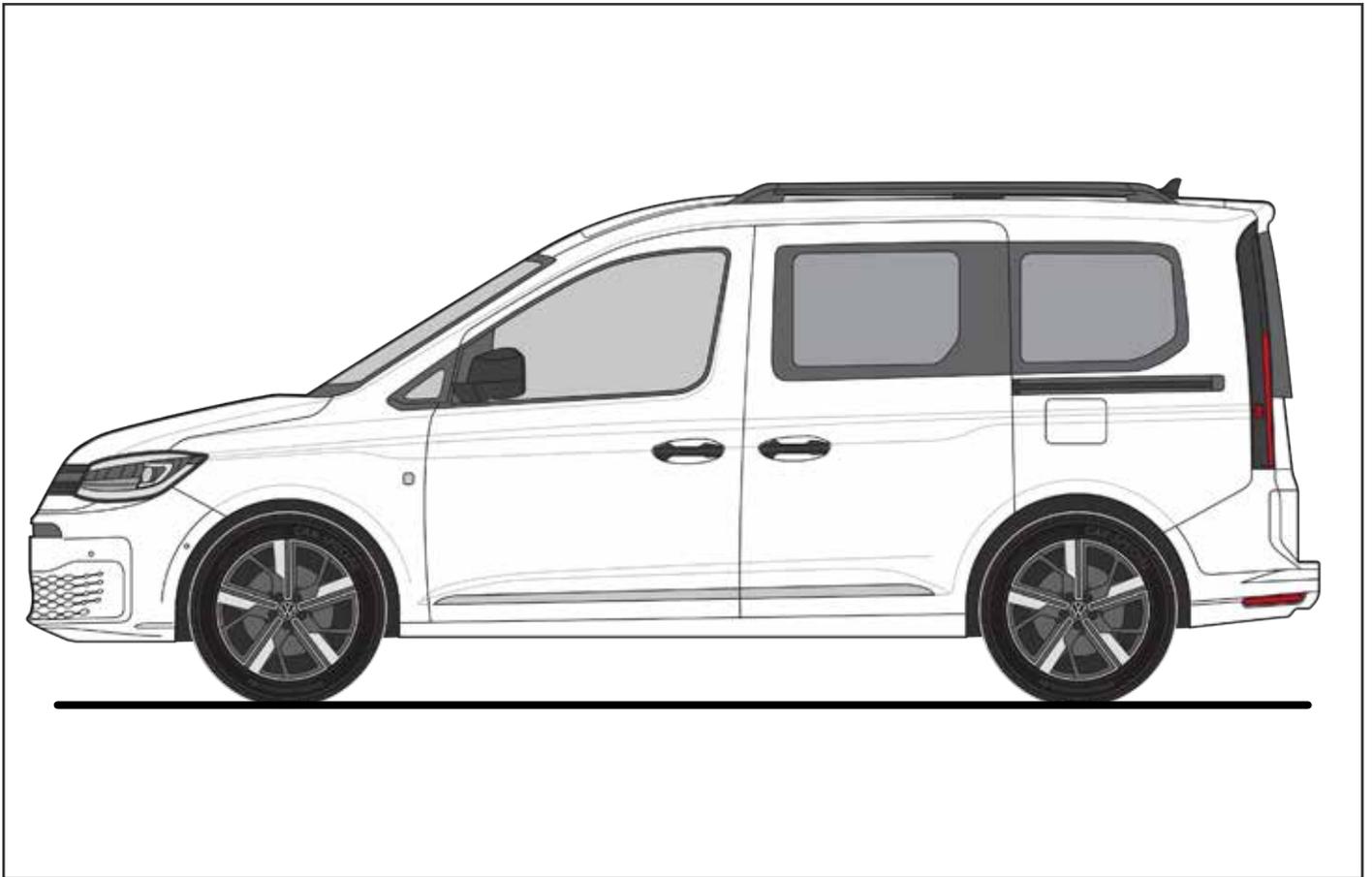
FOCACCIA GROUP
AUTOMOTIVE

VW CADDY WAV

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USER OPERATING INSTRUCTION

5354403 | Rev. 05 - 03/07/2023

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+ 39 0544 202344

ATTENTION

- It is recommended that you have a good understanding of both the stop mode and the operation of all commands.
 - Proper knowledge of the use, warnings and safety rules of the equipment and components and equipment that are part of the set-up and / or delivered together with the set-up.
 - **Before using each installed device, the operator in charge must read and fully understands all the warnings and standards listed in this manual.**
- Furthermore, operator has to be in psycho-physical conditions suitable for proper use and safe.**
- **Always check before each use that the equipment, where applicable, is accompanied by the respective user manuals and / or information sheets issued by the manufacturer: if there are any deficiencies or defects in the documentation, contact us immediately.**
 - **While the vehicle is running, the equipment, equipment and devices transported must be adequately anchored with the special straps, elastic bands or fasteners supplied.**
 - The manufacturer declines all responsibility for damage to persons or things, due modifications not permitted to devices and regulations not of served. Do not use the devices if there is a suspicion of malfunction. In these cases, contact Focaccia Group S.r.l. After Sales.
 - Only assigned operator for device use is authorized to stay nearby area of action, still remaining in a safe area.
 - Before driving the vehicle, each occupant must wear the seat belt correctly.
 - Ramp can be used only for wheelchair passenger transport: ramp cannot be used for objects lifting.
 - **Avoid NOT authorized ramp use: after each operation, close the ramp and check its safety conditions.**
 - **Before each use, check the correct operation of all safety devices.**
 - Before each ramp use check vehicle's brakes are activated and car is not on inclined ground.
 - Avoid hands close to ramp edges and closing points, during opening / closure.
 - **Before vehicle's driving, check wheelchair and its passenger are well secured in safety conditions.** Different way of wheelchair and passenger's securing are **NOT PERMITTED**. For further information, please contact Service Dep.
 - To withstand the forces exerted by the anchoring system in different driving conditions, a wheelchair with a structure that meets the relevant parts of the standard is recommended ISO 7176-19: 2008.

information

USEFUL CONTACTS - SERVICE:

What to ask to Focaccia Service Dep.?

- further information about vehicle's conversion use
- refer uncorrect functionality of conversion's parts, provided by Focaccia Group S.r.l.
- damaged parts warranty substitution request

Which information do you have to give to Focaccia Service Dep.?

- VIN - Vehicle Identification Number
- describe object which is broken / damaged or describe functionality problem
- send images or video about problem to be solved: more information Service Dep. has, quicker solution will be provided

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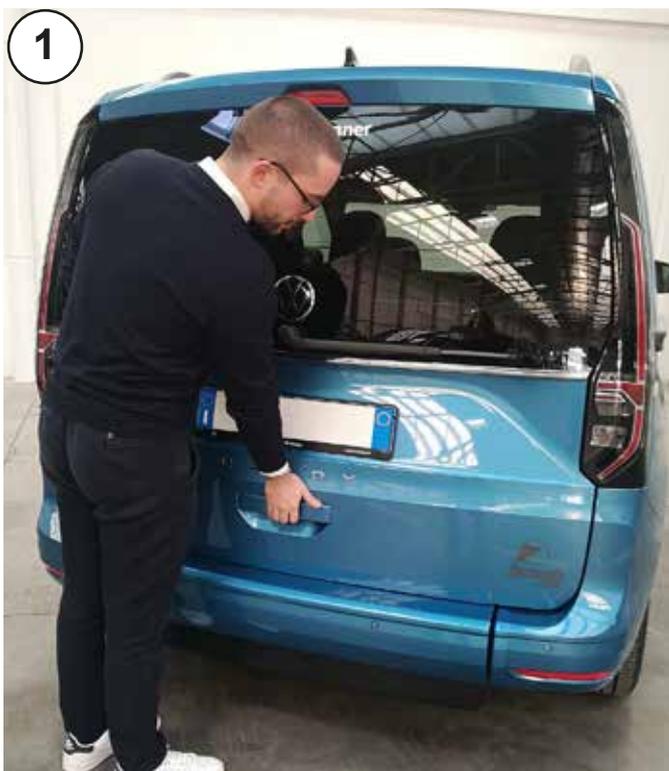
RAMP DEPLOYMENT:

- Park the vehicle on a hard standing level ground.
- Allow at least 2.5 Mtr clear distance behind the vehicle for the ramp to be deployed easily.
- Apply vehicle's handbrake.
- ➔ For rear door opening refer to vehicle's User and Maintenance Guide.

LOWERING THE RAMP:

For ramp lowering pull the knob on the left side of the ramp construction, and, using the built in handles simultaneously lower the ramp.

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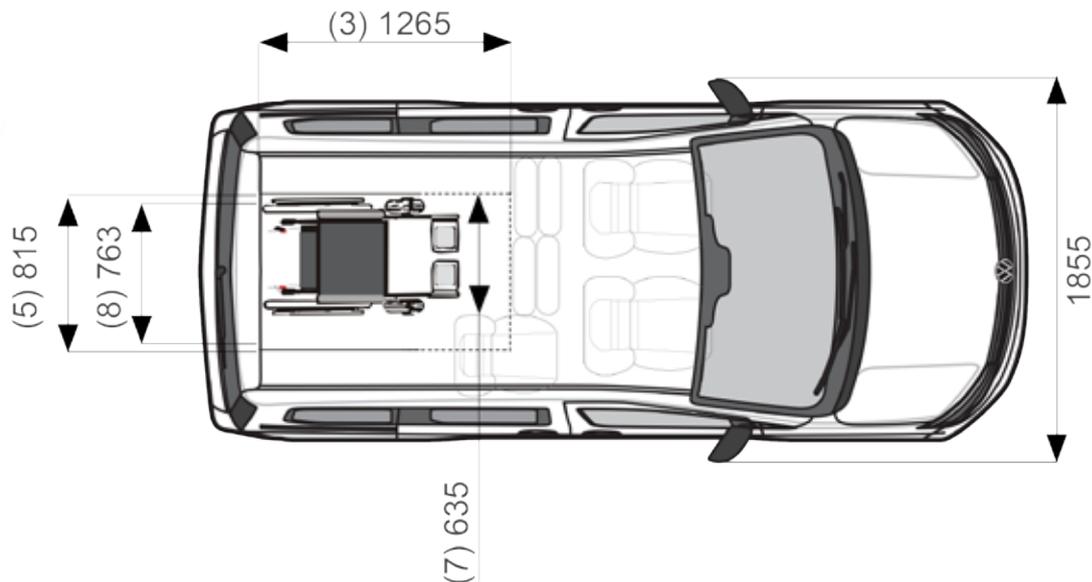
VEHICLE CONFIGURATIONS:

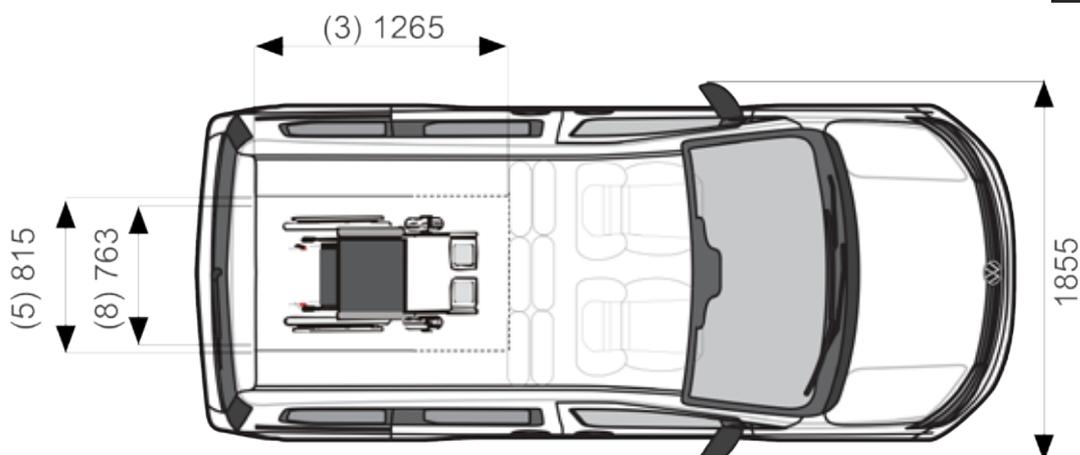
The converted vehicle can be used in the following configurations:

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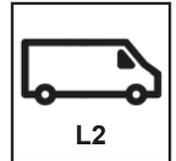
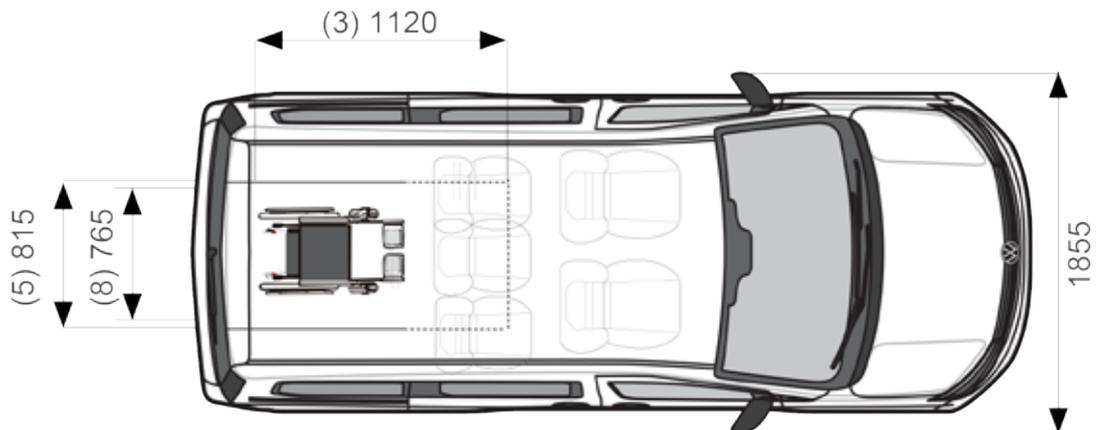
3+1 CONFIGURATION

Fold the seats and secure them as following details:

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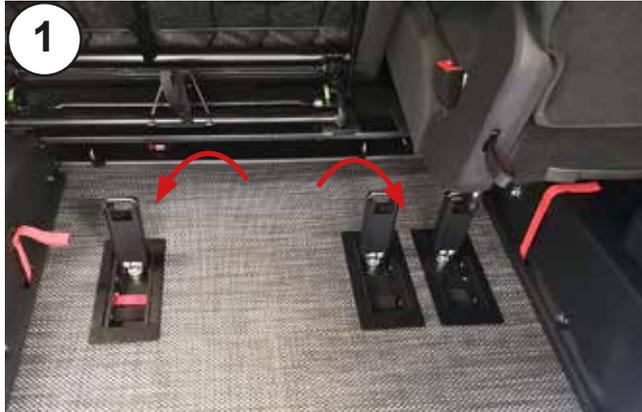
*Extract front retractors:
external left (E) and
central (C).*

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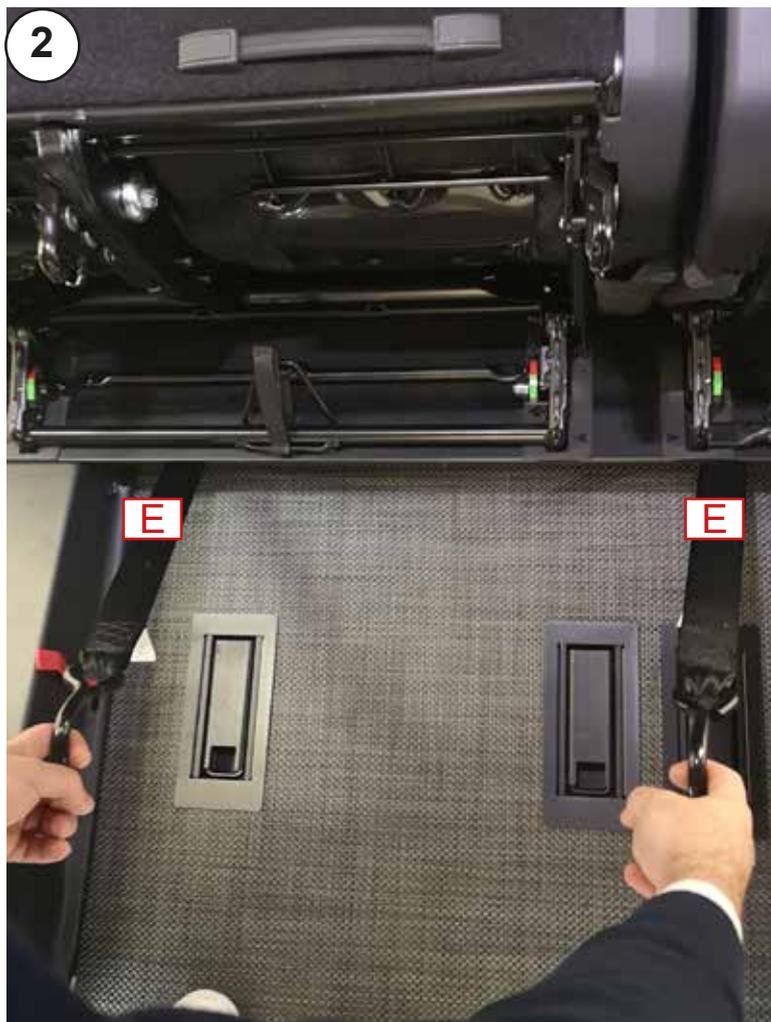
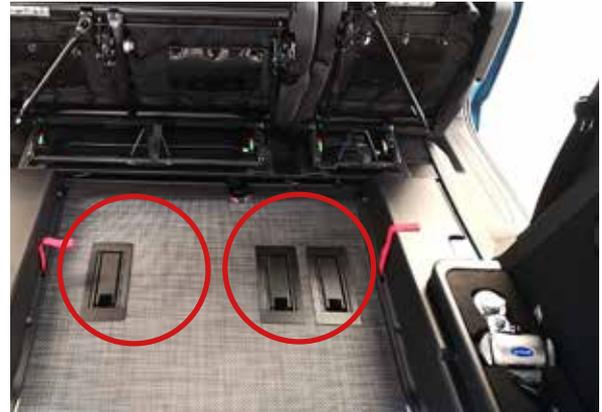
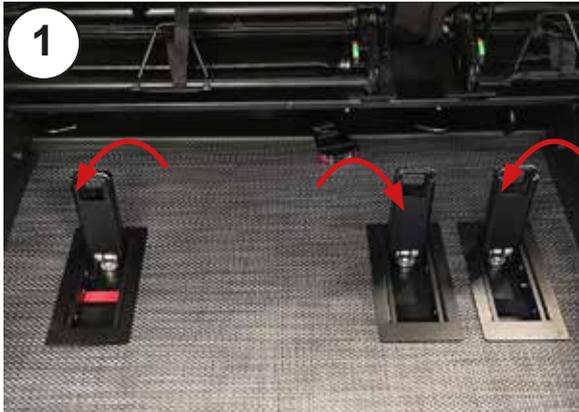
*Extract front retractors:
externals (E).*

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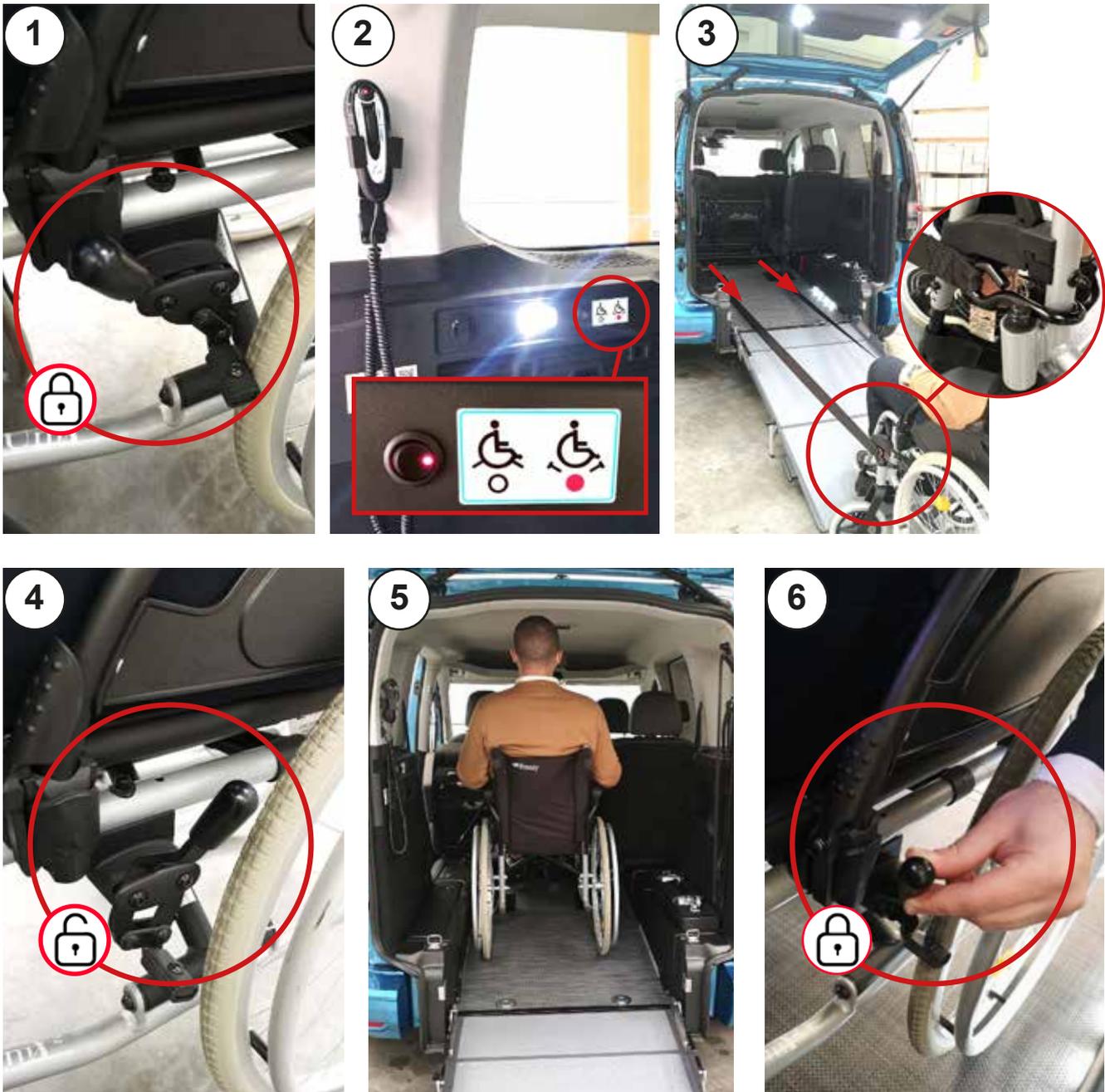
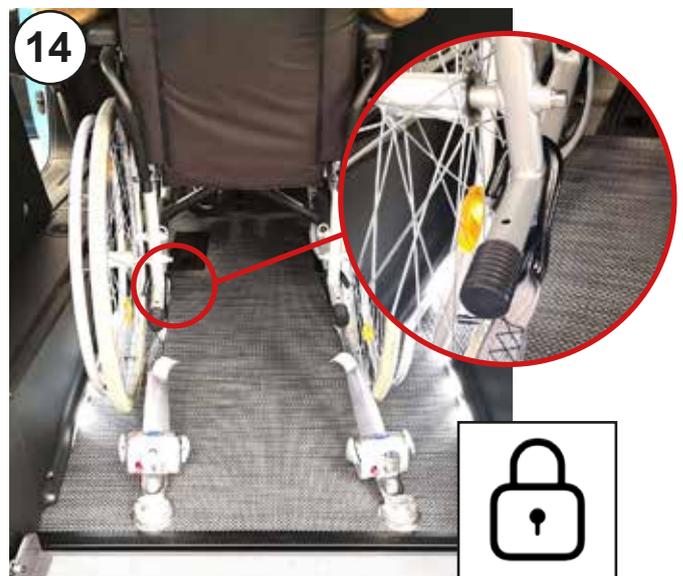
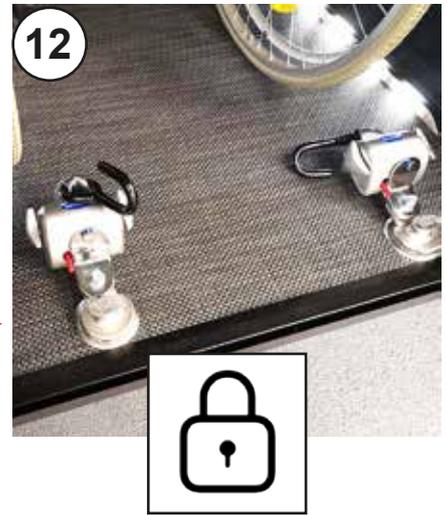
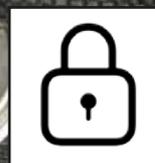
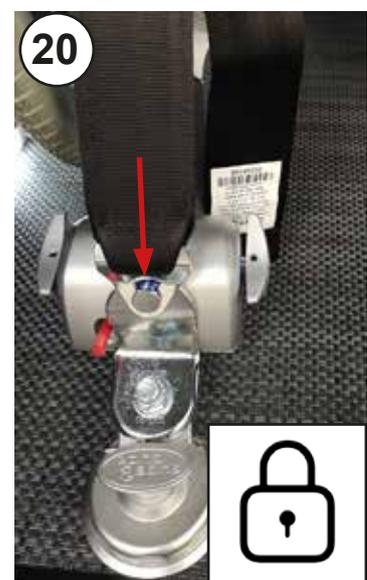
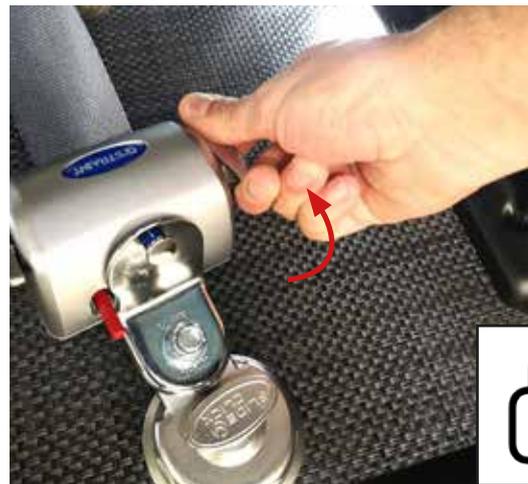
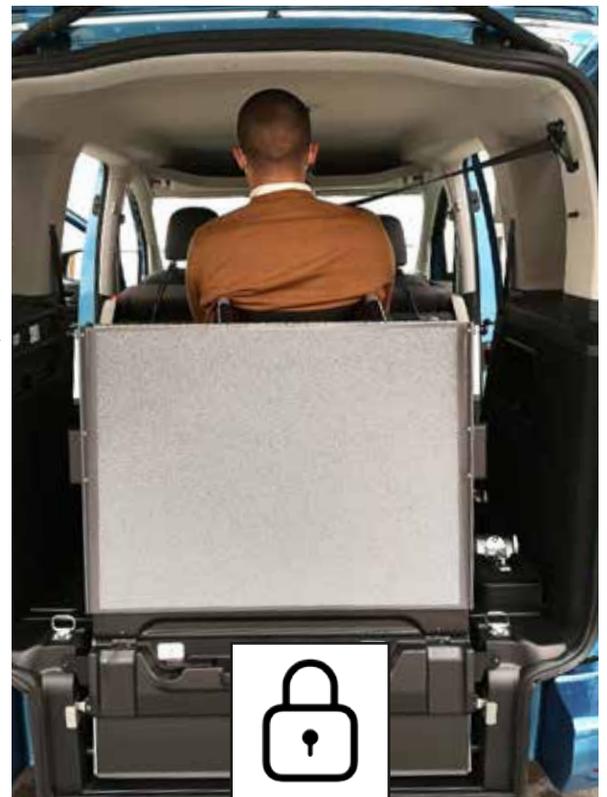
For person on the wheelchair vehicle's entrance, follow next details:

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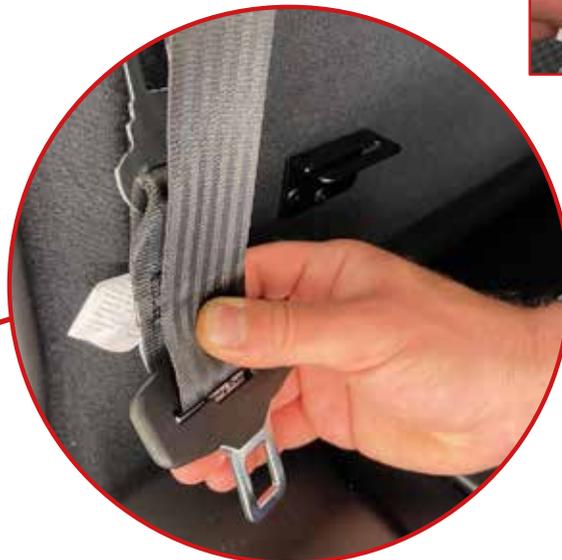
*Be careful to insert each seat belt into corresponding buckle.
To wear belt correctly it is necessary to position it as in **Ref. 5**.
Carefully read the instructions found on the safety belt **Ref. 3**.*

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*Pull strap and insert tab into the buckle **Ref. A**, insert tab into the buckle **Ref. B**.
Check that each buckle is properly secured by pulling on the strap.
If center seat display is off, make sure that passenger is in any case wear belt **Ref. 6** correctly.*

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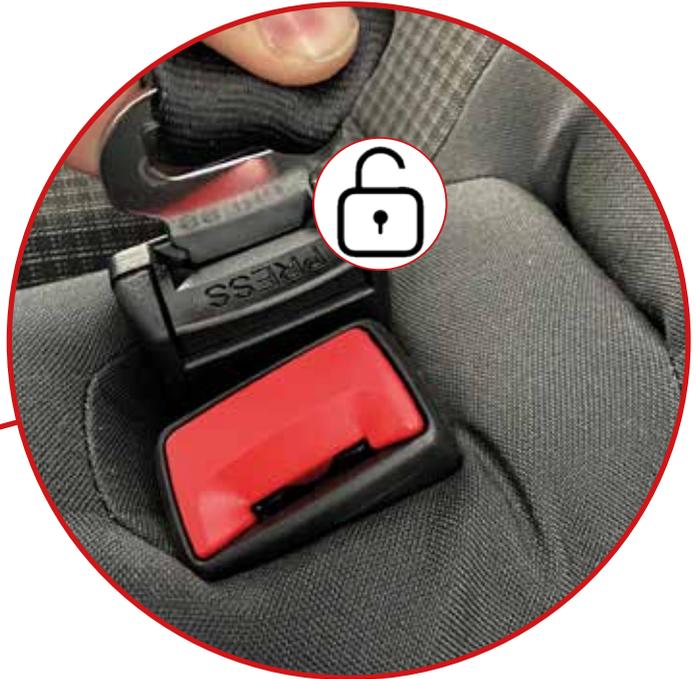
*In any case, make sure that the passenger wear belt **Ref. 6** correctly.*

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*When transporting passenger in a wheelchair, the seat belt must be positioned as in **Ref.11**.*

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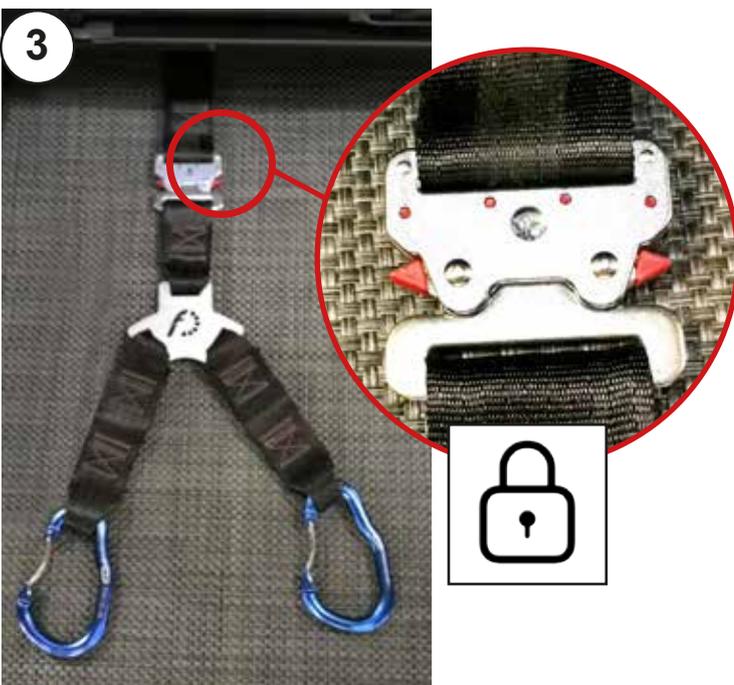
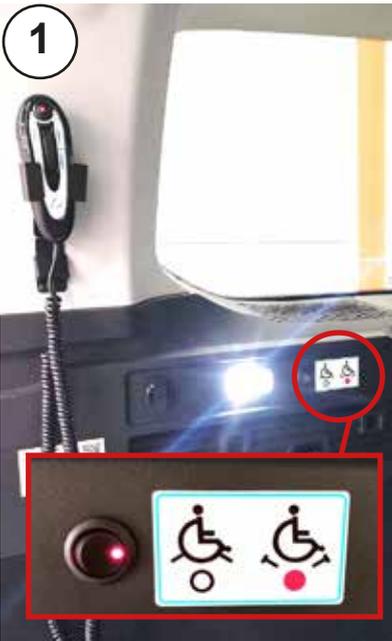
*To allow person on the wheelchair access using the F-Winch remote control, follow next details.
For passenger's securing, see the instructions in the previous paragraph.*

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Every maintenance operation has to be done only by authorized Focaccia Group S.r.l. workers.

It is fundamental that vehicle's conversion is checked frequently / maintenance has to be done regularly: maintenance intervals have to be planned depending on vehicle's use and years.

15.2. ORDINARY COMPONENTS MAINTENANCE:

It is strongly recommended to follow Manufacturer devices Use & Maintenance manuals, which are given with purchased vehicle.

15.3. CLEANING:

△ ATTENTION

Use only specific products for vehicle's cleaning.

- *Keep at least 60cm distance from vehicle's parts when washing it with high pressure car wash station.*
- *Do not use direct water jet into external wrap / electrical devices / plastic parts connection points.*

- *Do not use flammable liquids / alcohol / strong products / wax / petrol for conversion parts cleaning.*
- *Always use wet fabric: if it is necessary use neutral products.*
- *See following table for further information.*

- *During long period vehicle's stop, it is suggest to cover it with car carpet.*

MATERIAL:	MAINTENANCE:	HOW TO CLEAN:	FORBIDDEN OPERATIONS:
ABS Plastic parts and fiberglass	ORDINARY	Anti-dust tissue	DO NOT use abrasive products, alcohol, silicones or solvents.
	EXTRAORDINARY	Wet soft tissue with neutral products for internals cleaning	
Painted metals	ORDINARY and EXTRAORDINARY	Wet soft tissue with neutral products for internals cleaning; immediately clean surfaces after each cleaning process. Eventually use soft brush that DOES NOT damage surfaces.	DO NOT use tissues with synthetic fibres. DO NOT use abrasive or scratching products, acids, solvents or alcohols. DO NOT clean surfaces acting with high pressure.
Non slip surfaces	ORDINARY	Sweep regularly surfaces with nylon bristles broom.	DO NOT use tissues / papers because could leave small parts. DO NOT use metallic-hard bristles brooms / brushes. DO NOT use abrasive products, alcohol, silicones or solvents.
	EXTRAORDINARY	Only use vacuum for mud / dust / dirty removal. For dirty easy removal surfaces should be wet: then, use a broom / brush to clean.	
PVC parts	ORDINARY and EXTRAORDINARY	Only use vacuum for mud / dust / dirty removal. For dirty easy removal surfaces should be wet: then, use a broom / brush to clean. Internals cleaning products are suggested.	DO NOT use tissues with synthetic fibres. DO NOT use abrasive or scratching products, acids, solvents or alcohols. DO NOT clean surfaces acting with high pressure.
Polycarbonate	ORDINARY and EXTRAORDINARY	Clean surfaces with microfibre tissue and neutral soap. After cleaning, dry each cleaned surface.	DO NOT use tissues with synthetic fibres. DO NOT use abrasive or scratching products, acids, solvents or alcohols. DO NOT clean surfaces acting with high pressure.

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16. HOW TO REQUEST SPARE PARTS:

For spare parts request refer to Focaccia Group “Service” dep. or authorized center.

For spare parts request following information have to be given:

- VIN - Vehicle Identification Number ○ Year of production ○ Type of vehicle
- Spare part code ○ Quantity requested ○ Way of shipment

17. WARRANTY & RESPONSIBILITY:

For Warranty limits and terms see conditions listed while signing contract with Focaccia Group S.r.l.

- **Damages not covered by warranty:** *damages that occurs from natural disasters, car crash, electrical problems from connection to OEM cables, modifications of conversion or OEM parts not authorized, incorrect use of vehicle, maintenance done by not authorized workshops.*
- **Fixing operations:** *operations which cannot be done in place, can be done in authorized Focaccia Group S.r.l. workshops.*
- **Warranty loss:** *customers cannot request refund if vehicle has be used in a wrong way or things listed in previous point (“Damages not covered by warranty”) are done in not correct operation.*

Focaccia Group S.r.l. is responsible for safety, affidability and functionality of conversion, if:

- *vehicle’s users have strongly understood how to manage each described operation in correct way*
- *maintenance operations have been done as agreed during contract signing and only authorized spare parts have been installed.*

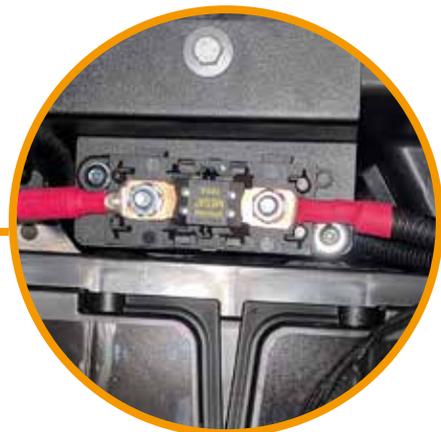
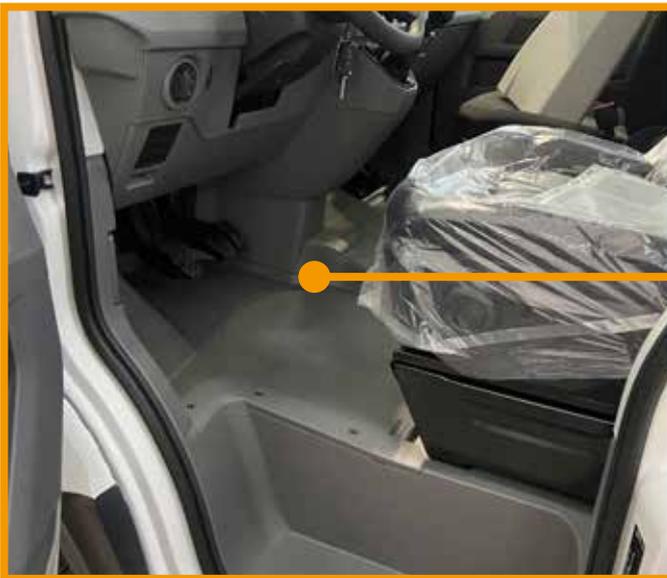
18. RECYCLING:

- *All components / materials have to be disposed as country recycling rules.*
- *Before recycling verify town regulamentations.*

Component	Material	Code	Where?
PALLET	Wood		Follow town recycling regulamentations.
CARDBOARDS	Cardboard		
FILM /PLURIBALL BAGS	Polyethylene		

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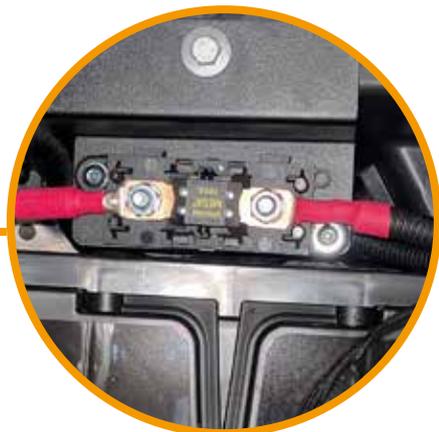
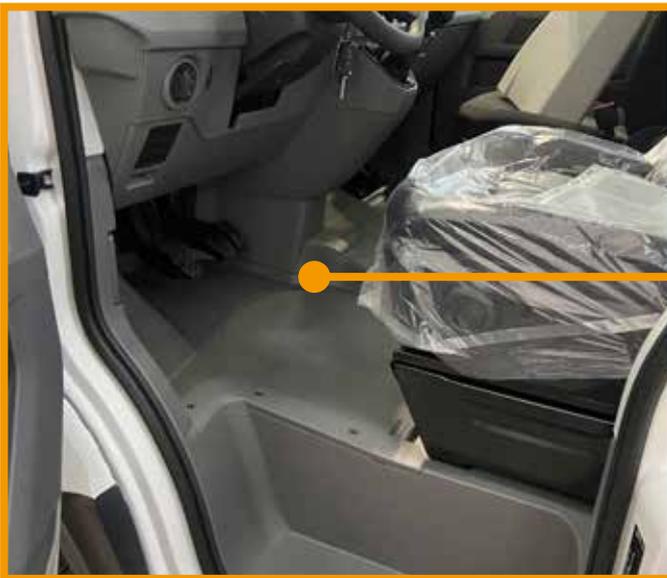
DRIVING CABIN FUSE POSITION

i information

IN CASE OF ELECTRICAL DEVICES NOT WORKING (such as front retractors):

- **Check general fuse:** this is located close to vehicle's battery, which can be in front bonnet or cabin area (under front seats) - it depends on how vehicle has been designed.

i information

For further information check "**Useful Contact - Service**" at the beginning of manual.

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i information

FOCACCIA GROUP S.R.L. WARRANTY ACTIVATION:

- Focaccia Group offers 2 years of warranty to components or parts that can present production flaws, after vehicle's defects.
- All fixing operations have to be done by authorized Focaccia Group S.r.l. workshops.
- Additional warranty options depends on what has been agreed during contract signing.
- For **warranty activation** scan QR-CODE:

21. MAINTENANCE:

i information

CONVERSION KIT MAINTENANCE:

- Correct and regular maintenance elongate product life, ensuring its efficiency and safety.
- After vehicle's first use, conversion kit has to be inspected periodically (at least once a year) by someone who's been charged for technical maintenance: it will be evaluate kit integrity and correct functionality of all parts.
- All moving parts (such as hinges, locking systems, springs, pistons, etc.) has to be periodically cleaned and lubricated with dedicated products.
- Every 6 months lubrication is recommended (these operations have to be done earlier, if vehicle's used in stronger conditions).
- After long period of vehicle's stop, before its use, check correct functionality and condition of all conversion parts.
- Workshop maintenance has to be signed from company who has take in charge these operations, signing following tables.

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